# THE QUE-U EXPERIENCE

#### SHUFIX



WHITE

Ultra fast white urethane glue. Very suitable for making foal extensions and urethane shoes.



#### SHUSIL



MEDIUM-SOFT

A20

Firm but soft acid free silicone pad material in three versions A10 - A20 - A30.

**SOFT** 

A10

MEDIUM

A30

#### SHUPACK



A50

Firm but soft urethane pad material in 2 durometers A30 Soft and A50 Regular or CS.

SOFT A30



CS A50

#### SHUGUARD



A60

Firm A60 urethane pad material very suitable for supporting the inner sole.



#### PRE FIX



**ACTIVATOR** 

**CA GLUE** 

PRE FIX is a system to bond a GLUSHU to the foot just to keep it in place.



#### SHUBOND













BEIGE **FAST SET** 



Ultra-light, 35% lighter than standard materials. Optimal softness and support. SUPER SOFT

SHUDIM A15 (Blue): Soft and resilient for gentle shock absorption. Easy to mix and shape.

SOFT A25

SHUDIM A25 (Pink): Balanced firmness for versatile protection. Non-irritating and easy to cut.

MEDIUM

SHUDIM A35 (Green):



GLUSHU

**GLUSHU** 

GLUSHU is a flexible cuff bonded to a specially designed aluminium horse shoe.



Acrylic glue for bonding all kinds of horseshoes and hoofwall repairs. Bonds steel, aluminium and even urethane shoes.

**BLACK** 

SHUDIM Air A20:

AIR A20

Firm support for demanding uses. Easy to handle and customize.



# CATALOG

# SHUPACK

Firm but soft urethane pad material in 2 durometers A30(soft) - A50 - A50(CS). 250cc cartridges fitting in caulking guns. Offers protection, cushioning and support for the hoof sole. Fill up to ground level for support of foundered and laminitis cases. When used without pads, usage of meshes is recommended. Set time approx. 35 seconds. Temperature and humidity may affect the performance of this product.



SOFT	OEA
Product Number	200130250
Description	Soft shock absorbing pad material with adhesion
Cartridge	250cc
Color	Cristal Clear
Set time	35 seconds

	A50
Product Number	200150250
Description	Soft & supportive pad material with adhesion
Cartridge	250cc
Color	Cristal Clear
Set time	35 seconds

CS	A50
Product Number	200152250
Description	Soft & supportive pad material with adhesion and extra protection against thrush
Cartridge	250cc
Color	Green
Set time	35 seconds

# SHUGUARD

Firm A60 urethane pad material very suitable for supporting the inner sole. 250cc cartridges fitting in standard caulking guns. SHUGUARD is a supporting pad material which can be used on horses with shoes or barefoot. Thanks to the support on the sole it relieves the pressure on the wall and improves heel growth. Fill up to ground level for optimal support. Set time approx. 35 seconds. Temperature and humidity may affect the performance of this product.



	A60
Product Number	200160250
Description	Supportive pad material with adhesion
Cartridge	250cc
Color	Black
Set time	35 seconds

# SHUSIL

Firm but soft silicone pad material in 3 versions A10 (blue) - A20 (transparent) - A30 (purple). 250cc cartridges fitting in caulking guns. Offers protection, shock absorption and support for the hoof. Always use SHUSIL in combination with Shupads! SHUSIL does not contain acids or chemicals. Setting time 4 minutes, depending on weather conditions. Heating up SHUSIL before use speeds up setting time significantly. Temperature and humidity may affect the performance of this product.



SOFT	A10
Product Number	210105250
Description	Soft Silicone pad material
Cartridge	250cc
Color	Blue
Set time	4 minutes (heated 2 minutes)

MEDIUM-SOFT	A20 TRANSPARENT
Product Number	210103250
Description	Medium Soft Silicone pad material
Cartridge	250cc
Color	Transparent
Set time	4 minutes (heated 2 minutes)

MEDIUM	OEA
Product Number	210109250
Description	Medium Silicone shock absorbing pad material
Cartridge	250cc
Color	Purple
Set time	4 minutes (heated 2 minutes)

# SHUDIM

Lightweight impression material for shock absorption and protection. Easily mixable and shapeable by hand.

Standard SHUDIM available in 2x660gr / 2x1.5lb or 2x2.5kg / 2x5.5lb and durometers A15 (blue) - A25 (pink) - A35 (green). The ultra light SHUDIM Air is available in 2100 cc in durometer A20 and weighs 35% less than any standard impression material in the industry.

SHUDIM is a non irritating product. Easy to cut which makes it applicable in many situations. SHUDIM does not contain acids or chemicals. Temperature and humidity may affect the performance of this product. Heating up SHUDIM before use speeds up setting time significantly.









ULTRA LIGHT	A20
Product Number	250114140
Description	Ultra Light Silicone pad material mixed by hand
Bucket + tubes	2100 cc + 3x23cc
Color	Orange
Set time	4-5 minutes (heated 2-3 minutes)

EXTRA SOFT	A15
Product Number	250105660 (660 gr/1.5 lb)   250105025 (2500 gr/5.5 lb)
Description	Extra Soft Silicone pad material mixed by hand
Jar / Bucket	2x660gr / 2x1.5lb or 2x2.5kg / 2x5.5lb
Color	Blue
Set time	4-5 minutes (heated 2-3 minutes)

SOFT	A25
Product Number	250106660 (660 gr/1.5 lb)   250106025 (2500 gr/5.5 lb)
Description	Soft Silicone pad material mixed by hand
Jar / Bucket	2x660gr / 2x1.5lb or 2x2.5kg / 2x5.5lb
Color	Pink
Set time	4-5 minutes (heated 2-3 minutes)

MEDIUM	A35
Product Number	250107660 (660 gr/1.5 lb)   250107025 (2500 gr/5.5 lb)
Description	Medium Silicone pad material mixed by hand
Jar / Bucket	2x660gr / 2x1.5lb or 2x2.5kg / 2x5.5lb
Color	Green
Set time	4-5 minutes (heated 2-3 minutes)

# SHUFIX

Urethane adhesive that is very suitable for making foal extensions and urethane shoes. Also suitable for retouching or rebuilding heels, making medial/lateral corrections and for smaller hoof repairs. The hoof should be clean and especially dry. 250cc cartridges fitting in caulking guns.

Set time between 30 - 60 seconds, workable after about 5 minutes. Temperature and humidity can influence the performance of this product.



WHITE	
Product Number	150101250
Description	Instant crack repair super fast setting / foal extensions
Cartridge	250cc
Color	White
Set time	30-60 seconds

# SHUBOND

Acrylic glue for bonding horseshoes and hoofwall repairs. Available in 150cc and 420cc black and beige. Fits in standard caulking gun using an adapter or just use the Glue-U dispenser for 150 cc cartridges. 420cc only usable with special gun. Bonds steel, aluminium and even urethane shoes. Suitable for repairing large cracks and creating extensions.

Initial set time: 90-120 seconds (open time). Weight-bearing in 4-5 minutes (even in cold conditions). Temperature and humidity may affect the performance of this product. Heating up SHUBOND before use speeds up setting time significantly.



BLACK	
Product Number	100102150 / 100102420
Description	Glue-on-shoe / Hoof repair
Cartridge	150cc / 420cc
Color	Black
Set time	4-6 minutes / weight bearing 7-8 minutes. Heated; 120-180 seconds

BEIGE	FAST SET
Product Number	100114150 / 100114420
Description	Glue-on-shoe / Hoof repair
Cartridge	150cc / 420cc
Color	Beige
Set time	150-240 seconds / weight bearing 4-6 minutes. Heated; 90-120 seconds

# PRE FIX

Pre Fix is a system to bond a glue-on shoe to the foot just to keep it in place. It is a 2 component super glue which will bond the GluShu to the hoof in seconds. This allows you to do the actual glueing of the shoe with the foot on the ground. Pre Fix is very useful if you have a very nervous horse or a horse that is unable to stand on three legs (laminitis or injury). Pre Fix is no structural glue. It is temporarily only to keep the glue-on shoe in place for the time you need until the actual glue is set. One component is the activator spray, the other component is a transparent gel in a squeeze bottle.



ACTIVATOR	
Product Number	490100200
Description	Pre glue GluShu in combination with Pre Fix CA glue
Cartridge	200cc
Color	Transparent
Set time	15 sec

CA GLUE	
Product Number	490101150
Description	Pre glue GluShu in combination with Pre Fix Activator
Cartridge	150 gr
Color	Transparent
Set time	15 sec

# **CARTRIDGE**STORAGE

#### DO'S

- ✓ Protect from freezing.
- ✓ Store ShuBond at approx. 10 °C / 40 °F.
- ✓ Store Poly-urethanes and silicones at a temperature of approx. 15°C / 60°F.
- ✓ Store the products at a dry place.
- ✓ "First in First out" use.
- ✓ Keep cartridges upright.
- ✓ Possibility to use mixing tip as cap.

#### DONT'S

- X Store the product in constant changing temperatures.
- X Put used cartridges back at the stock.
- X Open the cartridge for a couple of minutes.
- X Leave cartridge open after use.

# CARTRIDGE

#### DO'S

- ✓ Check the level of both components.
- ✓ Make sure both canals are free.
- ✓ Before actual use, first press just a little bit in a cup to ensure proper mixing before installing the mixing tip.
- ✓ Make sure the product is mixed properly before applying on the foot.
- ✓ Work in open well ventilated space.
- ✓ Wear protective clothing + gloves.
- ✓ Trim the hoof as normal. Trimming is the most important part of your job.
- √ While glueing, keep the foot up untill glue is completely cured.(setting time depends on temperature and humidity)

#### DONT'S

- X Do not smell or breath-in the product directly.
- X Contact with skin. Product might cause allergic reactions.
- X Don't use on wounds or open skin.
- X Use too much glue at once. More glue is more heat.
- X Avoid glue on the sole and frog.
- X Do not heat up cartridges in hot water. Air heating only.

# **GLUSHU**GLUE-ON HORSESHOE

GluShu is a flexible cuff bonded to a specially designed aluminium horse shoe. This gives the horse a light, hoof supporting, cushioned shoe which eliminates any stress or damage to the horse's hoof and is easy to apply. GluShus are non invasive, completely eliminating the risk of hoof injury due to nailing. Easy and fast to apply.

Uses a fast acting glue. We strongly advise you to use ShuBond glue which is specifically designed to be used with GluShu. The cuff allows the shoe to be glued in stages. No need for any special equipment. Simple sizing system & large range of sizes. No alteration needed. Eliminates the risk of stress when nailing into a thin white line or poor hoof wall. Cushions the feet. The cuff can be used to hold topical medication in place. Prevents further damage to hoof wall.

# GluShu Kit

SIZE	PRODUCT NUMBER
5x0 Front	410290050
4x0 Front	410290040
3x0 Front	410290030
00 Front	410290020
0 Front	410290010
1 Front	410290001
2 Front	410290002
3 Front	410290003
4 Front	410290004
4x0 Hind	410291040
3x0 Hind	410291030
00 Hind	410291020
0 Hind	410291010
1 Hind	410291001
2 Hind	410291002
3 Hind	410291003

# GluShu pair

SIZE	PRODUCT NUMBER
5x0 Front	400290050
4x0 Front	400290040
3x0 Front	400290030
00 Front	400290020
0 Front	400290010
1 Front	400290001
2 Front	400290002
3 Front	400290003
4 Front	400290004
7 Front	400290007
8 Front	400290008
4x0 Hind	400291040
3x0 Hind	400291030
00 Hind	400291020
0 Hind	400291010
1 Hind	400291001
2 Hind	400291002
3 Hind	400291003

# **GLUSHU**MINI GLUSHU (FOAL SHOE)

Mini GluShu is a smaller version of the standard GluShu applicable for as well front as hind feet. This glue-on shoe is a very easy to use glue-on shoe for foals and Shetland ponies. It is a versatile shoe with many options. It can easily be combined with Shufix for any required extension.

For users only applying Mini GluShus we also offer an application kit including all you need to glue on about 5 sets of Mini GluShu.

#### **APPLICATION KIT**

1x ShuBond 150cc1x Pre Fix set1x Adapter10x Mixing tip

PAIR SIZE	PRODUCT NUMBER
75 mm / 2.95"	400290075
85 mm / 3.35"	400290085
95 mm / 3.74"	400290095

PRODUCT	PRODUCT NUMBER
Application kit	410290000

# **SHUPADS**

The base for silicones and impression materials. These pads are available in 4 different types. M and XL flat, L flat and wedge with frog support. It's easy to cut the pads and fit to every hoof. Use our size chart to determine the size closest to what you need. The high quality urethane material ensures no limitation for the hoof mechanism and keeps pad materials in position.

Experiences proved that Glue-U Shupads are extra durable and can be used for a long period. Every Shupad is designed with self closing injection holes. So filling up the hoofs becomes easier with Glue-U Shupads.

#### **FLAT FLEX MEDIUM**

Product Number	810050010
Description	Flat pad with 4 filling holes
Volume	10 pcs (Medium)
Color	Black
Dimensions	150 x 155 mm 5.9 x 6.1"

#### FROG SUPPORT FLEX LARGE

Product Number	810060010
Description	Flat pad with multiple filling holes and frog support
Volume	10 pcs (Large)
Color	Black
Dimensions	165 x 165 mm 6.5 x 6.5"

### **FLAT FLEX EXTRA LARGE**

Product Number	810070010
Description	Flat pad with 4 filling holes
Volume	10 pcs (Extra Large)
Color	Black
Dimensions	180 x 180 mm 7.08 x 7.08"

#### **WEDGE LARGE**

Product Number	810060006	
Description	Wedge pad with multiple filling holes and frog support	
Volume	6 pcs (Large)	
Color	Black	
Dimensions	165 x 165 mm 6.5 x 6.5" 3° wedge; 10 mm 0.4"	

# **GLUSHU FROG SUPPORT**

The Glushu Frog Support offers targeted relief, support and protection for the horse. This insert is designed to address various hoof concerns in both front and hind hooves. These inserts are available in 11 different sizes corresponding the Glushu size.

#### **Applications:**

- Horses with contracted frogs: Helps expand and support the frog, promoting proper blood flow and function.
- Sore or bruised hooves: Protects sensitive tissues and aids in healing.
- Laminitis recovery: Provides crucial support during the sensitive recovery process.
- Hard terrain: Offers additional comfort and protection against wear and tear.



#### **FROG SUPPORT**

Product Number	430290075
Description	Universal Frog Support
Color	Black

PAIR	PRODUCT NUMBER
75	430290075
85	430290085
95	430290095
4x0	430290040
3x0	430290030
2x0	430290020
0	430290010
1	430290001
2	430290002
3	430290003
4	430290004







# **ADAPTER**

Product Number	490002001
	To fit 150cc SHUBOND in a standard caulking gun of universal dispenser
Volume	1 pc

### DISPENSER FOR SHUBOND 150CC

Product Number	827100150
Description	150cc dispenser
Volume	1 pc

## DISPENSER FOR SHUBOND 420CC

Product Number	827100420
Description	420cc dispenser
Volume	1 pc







## **UNIVERSAL DISPENSER**

Product Number	820002250
Description	150 & 250 cc dispenser
Volume	1 pc

## DISPENSER FOR 200CC

Product Number	827000200
Description	Dispenser
Volume	1 pc

## **DISPENSER 210CC SIDE BATTERY**

Product Number	820102210
Description	Battery dispenser
Volume	1 pc







### UNIVERSAL BATTERY DISPENSER BLACK 150/250/310 CC

Product Number	820102250
Description	150/250/310 cc battery dispenser
Volume	1 pc

#### BATTERY DISPENSER BLACK 420CC

Product Number	820102420
Description	Battery dispenser
Volume	1 pc

### ACCU DISPENSER 200/250/420CC

Product Number	820102001
Description	Accu dispenser
Volume	1 pc







# **DRAWKNIFE**

Product Number	490000002
Description	Knife to remove GluShu
Volume	1 pc

# SELF ADHESIVE FOAM BOARD

Product Number	810004001 / 810004050
Deceription	Foam board with adhesion 199 x 199 mm / 7.85 x 7.85"
Volume	1pc / 50pcs

### HAND WRAPPING FOIL

Product Number	890503001
Description	Hoofwrap foil
Volume	1 pc







# WRAPPING FOIL HOLDER


Product Number	890512001
Description	Hoofwrap foil holder
Volume	1 pc

# HEATER 250CC / 200CC / 420CC

Product Number	890312150 / 890312160 / 890312420
Description	Glue heating box 12V, 110V & 220V
Volume	1 pc

### MESH

Product Number	892002010
1)Ascrintion	Mesh for SHUPACK 2500 x 200 mm / 98.5 x 7.9"
Volume	1 pc

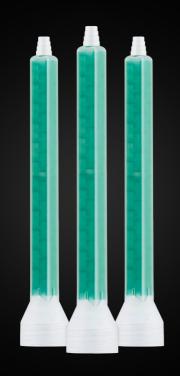



Product Number	610007010
Description	Green square mixing tip
Volume	10 pcs




## MIXERTIP 200 CC

Product Number	640107012
Description	Green 200 cc mixing tip
Volume	12 pcs



## SHUCLEANER

Product Number	490000750
Description	Aceton free hoof cleaner
Volume	1 pc

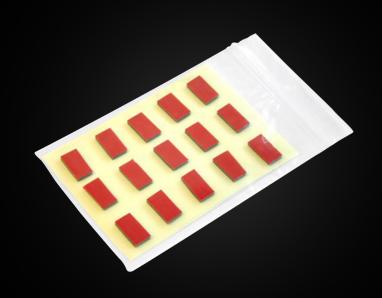



### **SPACERTAPE**

Product Number	891002010
	Special tape to keep space
Description	between the hoof and
	horseshoe
Volume	15 pcs

## SHUBOND DYE

Product Number	100102020
DACCHINTIAN	Black pigment to dye ShuBond beige to black
Volume	20 cc

# CONTACT

**GLUE-U ADHESIVES** 

Glue-U Adhesives BV Droogdokkeneiland 8 5026SR Tilburg The Netherlands

www.glue-u.com info@glue-u.com sales@glue-u.com +31 (0)13 545 31 18